

EasyStand[®] Magician[®] Original



The Way Kids Like to Stand



The EasyStand Magician Original is a sit to stand stander for kids from 3'-4'6" (91-137 cm) and up to 100 lbs (45 kg). The smooth planer seating system allows for easy independent and assisted transfers.

To use the Magician, simply transfer the child from their wheelchair into the seat of the stander. After securing the kneepads and the table, depress the foot button to lift the child to the desired standing position using the gas cylinder lift.

The sit to stand design provides a supportive transition to standing, reducing anxiety for the child. Even when a child is unable to stand fully upright, they can safely stand partially upright, while being fully supported. The stander can be adjusted to fit many different kids in multi-user environments, or grow with your child at home.



Shown with options

Anthony Sandstrom-Lee
Cerebral Palsy
42" (104 cm)

Magician Specifications

| | | |
|------------------------------------|-------------------|----------|
| Height Range | 3'-4'6" | 91-137cm |
| Weight Limit | 100 lbs. | 45kg |
| Seat Depth Range | | |
| -from seat pivot | 10.5"-15.5" | 27-39cm |
| Seat/Transfer Height | | |
| -from floor | 19" | 48cm |
| Seat to Foot Plate Range | 9"-15" | 23-38cm |
| Table/Chest Pad Depth Range | | |
| -from seat pivot | 5"aft-6"forward | 13-15cm |
| Table Height Range | | |
| -from seat pivot | 11"-22" | 28-56cm |
| Chest Pad Size | 7.5"x6" | 19x15cm |
| Knee Width Range | | |
| -center to center | 5.5"-8.75" | 14-22cm |
| Kneepad Depth Range | | |
| -from seat pivot | 3.5"-10.5" | 9-27cm |
| Back Height from Seat | 11" | 28cm |
| Hip Guides Width Range | 6.5"-12" | 16-31cm |
| Frame Color | Black Powder Coat | |
| Standard Upholstery Color | Royal Blue | |
| Standard Desk Table Size | 20"x22.5" | 51x57cm |
| Standard Footprint | 22"x33" | 56x84cm |
| Weight of Basic Unit | 75 lbs. | 34kgs |

"Solid, comfortable, functional. It's helping strengthen our son's muscles to have him stand on his own. He likes being upright and we love the fact that it has wheels so we can move him from room to room with the family. Price, EVERYTHING is great!"

Raymond & Gina J., Parents - Corona, CA, USA



Standard Features



#7000 EasyStand Magician Original

The Magician Original comes standard with royal blue or ruby red upholstery, padded seat, kneepads, back, hip guides, foot plates, 4" locking casters, clear acrylic table with front pad and gas spring lift.

Tool-free Adjustments



Seat Depth



Back Angle



Hip Guides



Table/Front Pad



Kneepads



Foot Plates

Optional Accessories



#P7500M Mobile Option

Allows the child to self propel in the standing position. Includes locking push rims, belt drive and two front wheels. *Table size is 15"x20" (38x51cm); Mobile Footprint is 23"x35" (58x89cm)*



#P80636 Foot Straps (Pair)

Hold the feet in place with Velcro® and d-ring adjustment. *Straps fit foot height of 2.5" -4" (6-10cm)*



#P80615 Secure Foot Option (Pair)

Velcro® & d-ring adjustment uses a four-point tightening system to hold the feet securely in place. *Straps fit foot height of 2.5" -4" (6-10cm)*



#P80806 Independent Kneepads (Pair)

Accommodate knee contractures and leg length discrepancies. *Kneepad size is 5"x 5" (13 x 13 cm); width range center to center is 6"-12" (15 - 30 cm)*



#P80998 Angle Adjustable Table

Attaches to the existing table. Supports the child while they work at various angles from 10° to 45°.



#P5024 Table Swing-Away Sleeve

Makes assisted transfers easier by moving the table away from the child.

Not available with the mobile option.



#P51501 Positioning Belt

Velcro® & d-ring adjustment provides moderate support for the child. *Fits hip circumference of 23"-36" (58-91 cm)*



#P7020 Lateral Supports (Pair)

Aid in positioning the child who has limited trunk control. *Width range is 7"-15" (18 - 38 cm); pad diameter is 4" (10 cm) round*



#P51601 Chest Strap - Small

Velcro® and d-ring adjustment provides moderate upper body support. *Chest strap pad is 5"W x 19"L (13 cm W x 48 cm L); fits users with chest measurement ranging from 35"-52" (89-132cm)*



#P80631 Chest Vest - Small #P80632 Chest Vest - Medium

Neoprene® with camlock adjustment holds the child securely in position. *Small size is 9.5" L x 9" W (24 cm L x 23 cm W); medium size is 11" L x 9.5" W (28 cm L x 24 cm W)*



#P5070 Ratchet Handles

Can be used in place of existing knobs for ratchet motion and leverage. *Four small handles for table, two large handles for foot plates*



#P7050 Stroller Handles

Make pushing the stander and lifting the child easier. *Height from the floor in seated position is 37" (94 cm)*



#P80246 Seat Angle Locator

Helps therapists monitor standing progress. Attaches under seat with magnet.

#P80809 Tool Pouch (not pictured)



#P7030 Head Support

Works as a prompt to limit head movement. *Height range from seat surface is 13"-19" (33 - 48 cm); depth range from back is 1" aft - 3" forward (3 cm aft - 8 cm forward)*



#P80642 Universal Head Support Receiver

Adapts to Whitmyer Biomechanix, Stealth®, and Otto Bock® head supports.