


Required Tools:


-13mm open end wrench -Hex Wrench



1. Pull the cloth protection sleeve down to expose the lower tray tube bolt and remove black protective cap.

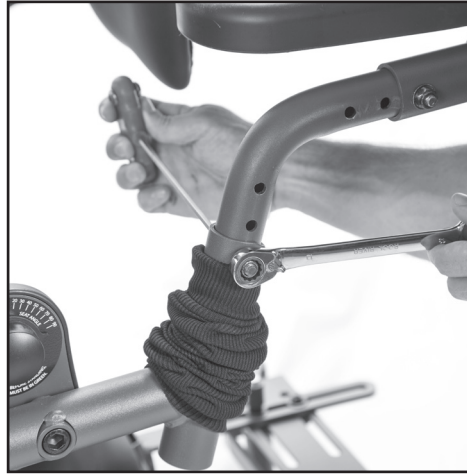


2. Use a hex wrench and 13mm wrench and remove nut and washer.

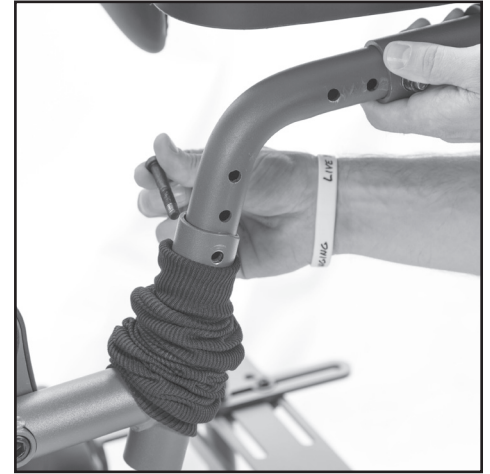


3. Remove the bolt.
Note: You may need assistance taking pressure off bolt by moving the tray.



4. Remove tray.

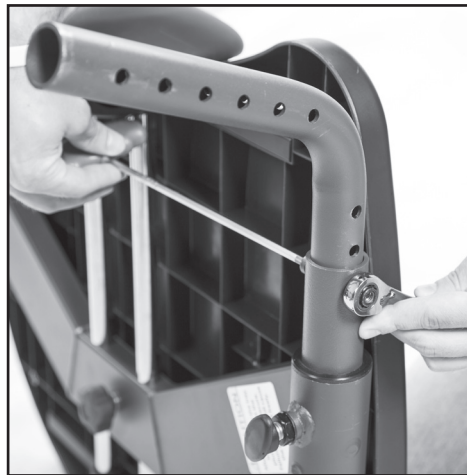

5. Use a hex wrench and 13mm wrench and remove nut and washer.

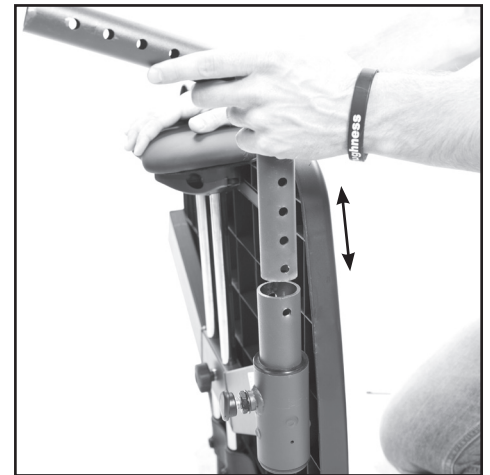


6. Place tray arm into desired position.



7. Insert bolt. Place washer and nut onto bolt and tighten securely. Place black protective cap onto nut.



8. Insert tray tube into the tray bracket at desired height.

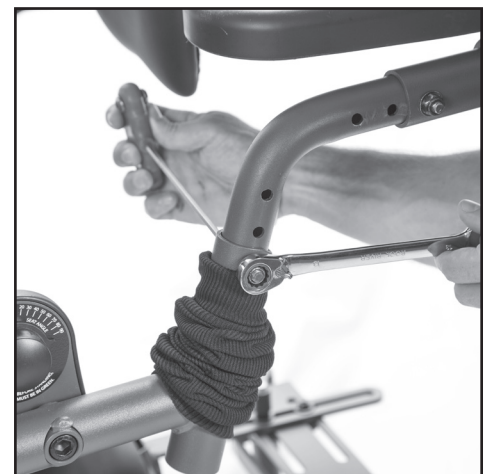

9. Using hex wrench and 13mm wrench tighten securely.



10. Place black protective cap onto nut.



11. Pull the cloth protection sleeve on the tray tube as shown.



12. Pull the cloth protection sleeve down to expose the bolt. Remove the black protective cap on the support arm.



13. Use a hex wrench and 13mm wrench and remove nut and washer.



14. Place into desired position. Use a hex wrench and 13mm wrench and tighten securely. Place black protective cap onto nut.



15. Use a hex wrench to adjust the support arm to level tray out.



16. Support arm must support the tray as shown.



The adjustable shadow tray has 5 height and depth adjustment slots.